

Registered trademark

# CELLU M6 Alliance

Please read the complete manual carefully before using your equipement.

© Copyright 2021. LPG, CELLU M6 and Alliance are registered trademarks of LPG Systems. Reproduction, even in part, is strictly prohibited.

> GU1602-EN Edition A2 - 01/2021



Congratulations on the purchase of your Cellu M6 Alliance<sup>®</sup>. This model represents the fruits of numerous years of experience in the design and production of cutaneous tissue treatment systems. You will experience fully the technical perfection and reliability of LPG Systems, which is the leader in this sector. With the aim to continuously provide satisfaction, your device is equipped with a specific software devised to ensure connection with the LPG Systems dedicated server. The data collected through this software will allow LPG Systems to give you better services in respect of support and maintenance.

This operator's manual contains the operating description, basic maintenance instructions to be performed periodically and the safety instructions. Your device is intended for use in the treatment of connective tissue. It should be used only by a professional who has attended the manufacturer's training provided by LPG Systems or an approved provider, if you live outside of France. If you have any doubts whatsoever concerning the operation or maintenance of your equipment, please do not hesitate to contact LPG Systems via the Customer Service Department of your distributor.

+33 (0)4 75 78 69 00

#### $\overline{2}$ ATTENTION

In order to respond better to customer requirements and expectations, LPG Systems is continuously researching ways of improving the design and quality of its products. This will explain the few minor differences between your equipment and the item described in this guide.

### → PACKAGE CONTENTS

- → One CELLU M6 Alliance<sup>®</sup> device
- $\rightarrow$  One Alliance 80 head
- $\rightarrow$  One Alliance 50 head
- $\rightarrow$  One TR30 head
- $\rightarrow$  One Ergolift<sup>®</sup> head
- $\rightarrow$  Two Ergolift<sup>®</sup> chamber (Lift20 and Lift10)
- $\rightarrow$  One electrical power cord
- ightarrow One unpackaging instructions and quick installation guide
- $\rightarrow$  One marketing set

Liste of accessories provided with your device :

Depending on the version you have, (see serial number on the nameplate), some protocols are not activated and their accessories are not provided. Accordingly, the paragraphs describing them do not concern this version (see table below).

In any doubt about the operation of your unit or to evolve into a complete model, please contact the Customer Service of LPG Systems or your distributor.

	Pack body	Pack face	Pack spa
Alliance 80	$\checkmark$		$\checkmark$
Alliance 50			$\checkmark$
TR30			
Ergolift			$\checkmark$
Ergolift chamber (Lift 10, Lift 20)			$\checkmark$
Unpackaging instructions and quick installation guide			
Cord			$\checkmark$
Marketing set		$\checkmark$	$\checkmark$

### $\rightarrow$ TABLE OF CONTENTS

### $\rightarrow~$ Description of the device

1.	DESCRIPTION OF THE DEVICE
2.	DESCRIPTION OF THE CONTROLS
3.	PRECAUTION FOR USE
4.	MAINTENANCE
5.	TROUBLESHOOTING
6.	TECHNICAL SPECIFICATIONS
7.	TREATMENT HEADS
8.	ENDERMOWEAR
9.	GENERAL WARRANTY CONDITIONS

### INTENDED USE


The CELLU M6 Alliance  $^{\odot}$  device is intended for aesthetic uses to body and face care. It can be used to:

- 1. Smooth the body's contour
- 2. Redesign of silhouette
- 3. Firming the skin
- 4. Reduce wrinkles
- 5. Enhance the skin aspect
- 6. Face contouring

#### 7 ATTENTION

The manufacturer reserves the right to amend the product technical specifications without prior notice. Any reproduction – even in part – is prohibited. All the illustrations in this operator's manual are non-binding.

### → THE CELLU M6 ALLIANCE<sup>®</sup> DEVICE

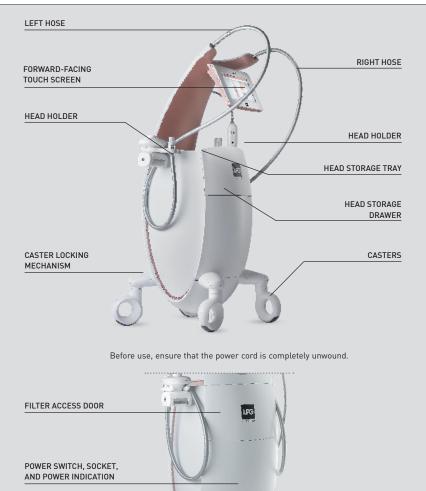


# $\rightarrow$ ACCES TO FILTER



ACCES TO FILTERS The filters are accessible via the back of the unit.

# $\rightarrow \ \text{CONTROL SCREEN}$

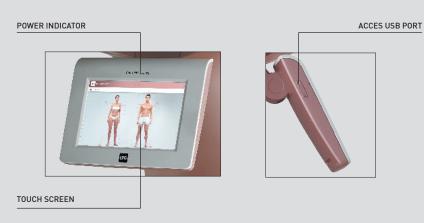

6

The unit can only operate if it is connected to the mains by its power cord and provided the ON switch has been actuated and the green voltage light is on. After switching on the unit, please wait a few seconds for the screen to display information.

IDENTIFICATION

PLATE

#### 7 ATTENTION


For detailed instructions on using the touch interface, refer to the touch interface operating manual received during training and available from customer service.

7

#### **ALLIANCE 80 HEAD** $\rightarrow$



# → ADAPTATER ERGOLIFT HEAD

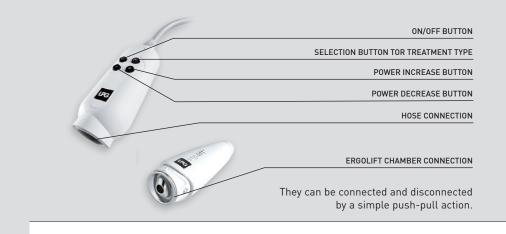

2

\_


CONTROLS

THE

ЧO

DESCRIPTION

### → ALLIANCE 50 AND TR30 HEADS

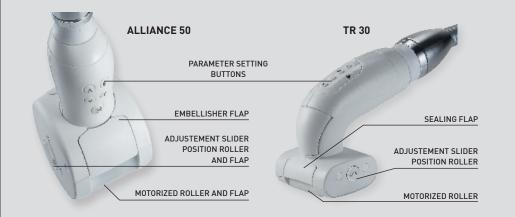

→ CASTERS

The CELLU M6<sup>®</sup> Alliance is equipped with locking casters. Please follow the procedure indicated below to lock or unlock the casters :





#### ↗ ATTENTION

In cases where the unit has not been moved for a long time, it is possible that marks may form on the ground where the casters are. This phenomenon is the result of a chemical reaction between the components of some floorings and those of the wheels of the CELLU M6® Alliance.

8

10

# → IMPORTANT SAFETY INFORMATION

All safety precautions must be observed whilst using electrical equipment. Please read all safety notices and precautions prior to use of the equipment.

### **DANGER** - TO MINIMISE THE RISK OF ELECTRICAL SHOCK:

- Always disconnect the equipment from the electrical supply outlet after use and before cleaning and maintenance.
- Check that the supply voltage of the unit indicated on the data plate complies with the mains voltage.
- The unit must be connected by the power cord1 supplied to a grounded outlet in accordance with current electrical standards. Electrical adapters must not be used with this equipment.
- Ensure that the unit is connected to a system with a differential protection against DC and AC.

### $\rightarrow$ IMPORTANT SAFETY INFORMATION

- Never use the equipment on a dusty, uneven floor, or in a moist atmosphere.
- Never use the equipment in the presence of aerosols or oxygen.
- Before disconnecting the unit from the electrical supply, set all controls to the 'off' position and unplug the unit. The disconnection of the unit is disconnecting the power outlet.
- It is prohibited to modify this equipment without prior authorisation from the manufacturer.
- It is prohibited to use components or spare parts non recommended by LPG Systems.
- Return the device to LPG Systems Service Center for examination and repair.

### $\rightarrow$ **PRECAUTIONS FOR USE**

#### ATTENTION: KEEP THESE INSTRUCTIONS.

Your device should be used on healthy skin. It is important to read and respect the following precautions and contraindications before using your device.

- $\rightarrow$  WARNING
- TO MINIMISE THE RISK OF BURNS, FIRE, ELECTRICAL SHOCK OR INJURY:
- The equipment must not be left unattended whilst connected to the electrical supply.
- Disconnect the unit from the electrical supply if it is not going to be used for a long period.
- Special attention is required whilst using the equipment with, or in the proximity of children or disabled persons.
- Never use the unit for purposes other than those recommended by LPG Systems. Only use the treatment heads supplied with your unit or recommended by LPG Systems.
- Never use the equipment if: The electrical power cord or outlet is damaged. The equipment does not function correctly. The equipment is damaged or has fallen or been dropped. The equipment has been exposed to excessive humidity.

- Do not move the unit by pulling the electrical power cord.
- Fully unwind the electrical power cord and keep it away from warm surfaces.
- Never use the equipment if the ventilation ports are obstructed. Ensure that the ventilation ports are kept clear of dust or other contaminants.
- Do not allow solid debris, liquid or other foreign bodies to fall or be sucked into the unit, as these could cause damage.
- Children shall not play with the deviceCleaning and user maintenance shall not
- be made by children without supervisionStop the device by using its main switch when it is not use.

- Never touch the client and the device's unprotected cables or connectors simultaneously.
- Never use the adapter as a treatment head or allow it to come into direct contact with the skin.
- Only use treatment heads supplied with your unit or recommended by LPG Systems.
- Do not use the treatment heads directly on the skin. Wear the treatment suit provided by LPG Systems, ENDERMOWEAR.
- LPG Systems will not be liable for any inappropriate use of the equipment.
- Improper use of the device can cause tissue damage or pain.
- The operator must be particularly attentive to the sensations felt by the person undergoing treatment.
- The operator must ensure that the parameters (intensity, sequentiality, differential...) are always adapted to the tissue being treated.

- Do not lean, rest, or sit on the unit.
- When crossing a threshold or step, we recommend moving the unit carefully by firmly holding the central arm monitor stand to avoid the risk of tipping.
- Do not use the USB connection during treatment.
- Do not operate the unit in unsuitable environmental conditions (see technical specifications).
- The power plug is used as disconnect device. The disconnection of the unit is disconnecting the power plug.
- Please position your device so that the mains unit is always accessible.
- Do not touch the client and the hose connectors simultaneous

#### ⊿ ATTENTION

Europe VII-H05VVF3G1,50-C19; Italy I/3/16-H05VVF3G1,50-C19; Switzerland 23G-H05VVF3G1.50-C19; UK BS13/13-H05VVF3G1,50-C19; Japan 498GJ-VCTF3X2.00-C19; USA, Canada, Mexico N5/15-SJT3X14AWG-C19 (connect to Hospital grade receptacle in hospital environnement). R

# → CONTRA-INDICATIONS

- Do not care open wounds, the eyes, intra-cavity areas, mucous membranes, the genitals or breasts.
- This device is not recommended for pregnant women. In the event of pregnancy, do not care the lumbar-abdominal region. Consult with a doctor with regard to this treatment.
- Do not care a client with an infectious disease, a growing tumour, a phlebitis, a wound, or an infected area.
- Do not care a client with skin cancer, a visible tumor, or other cancerous lesions. Consult with a doctor in cases where the client has a case history of tumours or is in remission.
- Do not care inflamatory areas or scars without medical advice and LPG technical training for the affected areas.
- Do not care people with circulatory problems without first consulting their doctor and without training in LPG technology for the affected areas.
- Do not care a client with an unexplained and persistent pain without medical advice and without training in LPG technology for the affected areas.
- Do not care a client after an invasive medical treatment without medical advice or surgeon who carried out the treatment and without training in LPG technology for the affected areas.
- To avoid bruising, exercise caution when determining a client's level of sensitivity and avoid use on clients taking anticoagulant drugs.

- Stop care immediately if the client experiences pain and consult a doctor.
- This device should not be used on all dermatoses, skin rashes, herpes, inflammatory or infectious acne or vitiligo.
- Because of the risk of interference, it is important that the professional ensures the client is not equipped with a personal medical device such as a pacemaker. If this is the case, the professional should obtain details about the device in question to ensure that any interference will not affect the correct use of the equipment.
- As this list is not exhaustive, always seek out medical advice in the event of doubt.
- For a more detailed list of indications and against indications, refer to training manuals.
- Don't care a client who takes anticoagulant drugs.

# → ELECTROMAGNETIC COMPATIBILITY

- The Cellu M6 Alliance® device uses RF power only for its internal functions.
- Therefore, its RF emissions are very low and are not likely to cause interference in a neighboring electronic device.
- The Cellu M6 Alliance<sup>®</sup> device can be used in all establishments, including domestic premises and those directly connected to the public low-voltage power supply network supplying domestic buildings.
- The Cellu M6 Alliance<sup>®</sup> device must be installed on a floor made of wood, concrete or ceramic tile. If the floors are covered with synthetic materials, the relative humidity should be at least 30%.
- The Cellu M6 Alliance® device must be connected to a power supply network whose quality is equivalent to that of a typical commercial or hospital environment.
- If the Cellu M6 Alliance<sup>®</sup> device user requires continuous operation during power outages, it is recommended to supply power from an uninterruptible power supply or battery.
- Portable and mobile RF communications devices must not be used too close to any part of the Cellu M6 Alliance®, including cables.
- Interference may occur in the vicinity of equipment marked with the following symbol:



#### 7 4

12

 $\underline{7}$  ATTENTION

This unit contains programs to help the operator obtain the best anticipated results for each case undergoing treatment. Under no circumstances may these programs be construed as a guarantee of successful treatment, which varies, depending on the morphology, physiology and eating behavior of each customer.

13

# → IDENTIFICATION RATING PLATE

Your unit is identified by a serial number shown on the rating plate. The rating plate also shows the permitted supply voltage for the unit. If you need to contact LPG Systems because of a technical problem, please indicate the serial number of your CELLU M6 Alliance<sup>®</sup>. This serial number provides information on the year and month of manufacture of your unit.

The letter indicates the year the device was manufactured. Z=2009, A=2010, B=2011, etc.

The two digits indicate the production month: 01=January; 02=February; 03=March; etc.

- This icon indicates that the unit was sold after August 13, 2006. In conformity with the 2002/96/CE directive, it cannot be thrown away with standard household waste but must be disposed of by means of recycling. When your device reaches the end of its useful life, it must be brought to an appropriate recycling center or returned to your dealer. By doing so, you help the environment by contributing to the conservation of natural resources and the protection of human health.
- This icon indicates that some specific warnings or precautions associated with this device are not on the label.
- This symbol means store the device somewhere protected from inclement weather.
  - This symbol indicates temperature limits.
  - This symbol indicates relative humidity limits.
  - This symbol means « danger: High Voltage »



This symbol indicates fragile, handle with care

 This symbol indicates keep it vertical

This symbol indicates not reverse

This symbol indicates athospheric pressure limits.

This symbol means always consult the accompagnying documents before using your device.

### $\rightarrow$ CLEANING THE UNIT

It is recommended that you clean your unit as often as possible, not only for reasons of hygiene and aesthetics but also because cleaning the unit will help keep it in a good state of repair and extend its useful life.

#### Using a vacuum cleaner with a fine nozzle, clean the following sections:

- Interior of the head storage drawer.
- Interior of the head storage tray.
- Interior of the filter access door.

#### Using a moist sponge, clean the following sections:

- All the external hoods.
- Hoses.
- The electrical power cord.

# Using a cloth soaked with a small amount of alcohol-free domestic cleaning product, clean the following sections:

- Control screen and control panel.
- Interior of the head storage drawer.
- Interior of the head storage tray.
- Interior of the filter access door.

### Using an antistatic cloth soaked or wipes, clean:

• The touch screen.

The maintenance of the care heads should be performed before the first use, see chapter «head of care».

4

ш

MAINTENANC

↗ ATTENTION

The identification rating plate is located on the underside, at the back of the unit. Identification rating plates may vary. The approved one is one on your machine.

#### ATTENTION ■

Proscribe aggressive products such as acetone, trichlorethylene or alcohol at 90°.

# $\rightarrow$ REPLACING THE FILTER CARTRIDGES AND FOAM

Your device contains two filter cartridges and one filter foam. These components guarantee the efficiency of your unit and prolongs its useful life.

UKS of the second

Ensure that they are changed as soon as the icon indicating a filter change appears on the screen (Fig 1) :



Access the 'filter change' menu as follows :

Select the 'maintenance' menu by pressing the icon indicated **(Fig. 2)**.

Select the 'filter' menu by pressing the icon indicated **(Fig. 3)**.

The 'filter change' screen indicates which filter requires changing **(Fig. 4)**.

Once the filter cartridge is replaced, the filter meter should be reste by pressing the icon indicated (Fig.5)



PRESS THIS ICON

FIG. 5

Replace the filter cartridges as follows:

Open the filter access door.
(Fig 1)

**2-3**. Unscrew and remove the filter cartridges and replace them with new ones. **(Fig 2 et Fig 3)** 



Remember to purchase new filter cartridges from the LPG Systems Customer Service Department so you always have a spare.



→ REPLACING THE FILTER CARTRIDGES AND FOAM



FIG. 3

4

MAINTENANCE

The device must never be used without a filter. If no filter is fitted, the device must be switched off.


### → CONNECTING / DISCONNECTING THE MOTORIZED TREATMENT HEADS

To connect the heads to the hose, follow the procedure below.

Position the locking ring in the locked position **(Fig.1)** 

Position the end of the hose so that the hose key is lined-up with the key slot of the treatment head connection **(Fig.2)** 

Push the hose into the treatment head connection until it cliks into place.

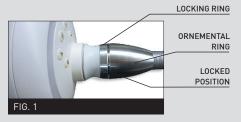



LOCKED

HOSE KEY

UNLOCKED

# $\rightarrow~$ Connecting / Disconnecting the adapter

To disconnect the heads to the hose, follow the procedure below.

Position the locking ring in the unlocked position (fig. 3) Pull the locking ring towards the hose (fig. 4) Carefully remove the hose by pulling it by the white ring (fig. 5)







UNLOCKED POSITION 4

MAINTENANCE

# → INSTRUCTION TO REMOVE USB PROTECTIVE COVER



Remove the access cover to the USB port by using the appropiate tool.

# $\rightarrow$ MAINTENANCE LOG SHEET

Replacement of filter cartridges: To be done when the warning message. Replacement of sealing flap: To be done when the flaps no longer treat the skin properly. They should be replaced after every 100 hours of use.

DATE	N° OF HOURS	OPERATION PERFORMED

### $\rightarrow$ REPLACING THE POWER CORD

If the power cord of your device is damaged, please contact LPG Systems Customer Service for a replacement.

LPG Systems Customer Service: +33(0)4 75 78 69 00

# → WHAT IF I'VE A PROBLEM ?

If your unit is not working properly, proceed with the following checks before calling Customer Services:

- Is the unit properly connected to a mains plug?
- Is the mains plug live?
- Is the ON switch lit up?
- Are the filter cartridges clean and correctly placed?
- Are the hoses properly connected?
- Are the heads of care properly connected ?
- Is the connected head of care connected corresponds to that shown on the screen?

Once these checks have been carried out and if the malfunction persists, please contact Customer Services of LPG Systems or the nearest authorized dealer, indicating the model of your unit and its serial number.

Service Client de LPG Systems: +33 (0)4 75 78 69 00

# $\rightarrow$ TECHNICAL SPECIFICATIONS

Net weight: Maximum set depression: Cooling:	
Electrical protection class: Wifi: Electrical features:	
Radiated Power:	Canal b : 12.46dBm Canal g : 9.33dBm Canal n : 9.48dBm BT 3.25dBm

Operating environmental features:

Ambient temperature:	+ 10 à + 30°C for normal use.
Ambient relative humidity:	30 à 75% without condensation.
Atmospheric pressure: 800 à 1050 hPa (For	use in a normally ventilated room)
Max altitude:	2500m

Environmental characteristics of transport and storage:

Temperature:	-20°C to +70°C
Ambient relative humidity:	
Atmospheric pressure :	50 hPa (For use in a normally ventilated room)

The Cellu M6 Alliance<sup>®</sup> device is marketed CE in compliance with the applicable texts and standards as part of the EMC 2014/30/EC and Low Voltage 2014/35/UE

HEADS OF CARE

# **CELLU M6 Alliance**°



DESCRIPTION OF ALLIANCE 80 HEAD	. 26
DESCRIPTION OF ALLIANCE 50 HEAD	. 28
DESCRIPTION OF TR30 HEAD	. 30
DESCRIPTION OF ERGOLIFT HEAD	. 31
DESCRIPTION OF MICRO-HEADS AND MICRO-NOZZLES	. 32
MAINTENANCE	. 33

Registered trademark

24

7

TRAITEMENT

TÊTES DE

# $\rightarrow$ DESCRIPTION OF THE ALLIANCE 80 HEAD



# $\rightarrow$ DESCRIPTION OF THE ALLIANCE 80 HEAD

### ADJUSTMENT OF ROLLER AND FLAP POSITION

Alliance head 80 has an adjustable stop with 2 positions for adjusting the distance between the motorized roller and of the damper.

When the cursor is in the horizontal position, the maximum mobility of the rollers is ensured.

When the cursor is in the vertical position, the minimum mobility of the rollers is ensured.



To change the adjustment slider turns it into the holding down to the desired position, as shown in the photo below:






26

HEADS

TREATMENT

# $\rightarrow~$ Description of Alliance 50 head

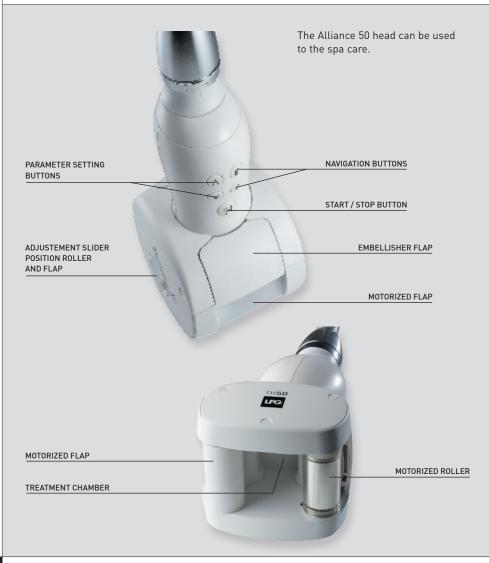

# → DESCRIPTION OF ALLIANCE 50 HEAD

### ADJUSTMENT OF ROLLER AND FLAP POSITION

Alliance head 80 has an adjustable stop with 2 positions for adjusting the distance between the motorized roller and of the damper.

When the cursor is in the horizontal position, the maximum mobility of the rollers is ensured.

When the cursor is in the vertical position, the minimum mobility of the rollers is ensured.



To change the adjustment slider turns it into the holding down to the desired position, as shown in the photo below:






28

TRAITEMENT


В

TÊTES

### → DESCRIPTION OF TR30 HEAD

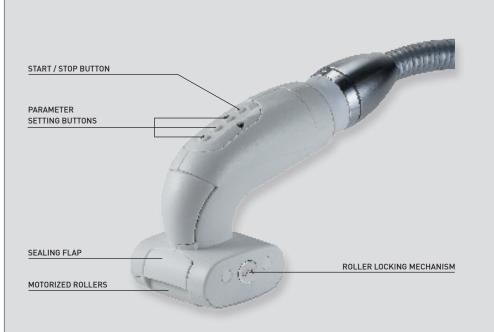

# $\rightarrow$ DESCRIPTION OF THE ERGOLIFT HEAD

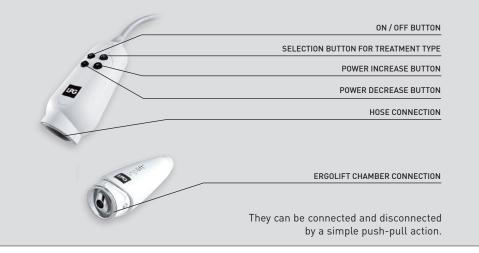

### → DESCRIPTION OF ERGOLIFT CHAMBERS

The Egolift chamber Lift 20 can be used to the care of large areas with fine tissues, and sensible areas. The Ergolift chamber Lift 10 can be used to the care to tight areas, eyes and lip contour, hands and figers.



30

/

HEADS

TREATMENT

#### MAINTENANCE $\rightarrow$

For reasons of hygiene, the maintenance of the care heads should be performed after each use, using antiseptic wipes impregnated with a bactericidal and fungicidal solution. Special attention must be given to the maintenance of the parts that are in contact with

the client.

### **THE ALLIANCE 80 HEAD**

- 1. Remove the sealing flaps (2 up flaps and 1 down flap) as showed in the pictures below (fig 1 to 4).
- 2. Thoroughly rub for at least one minute with the wipes as describe here below: a) Flaps and their housing (fig 5 to 7).
  - b) The casing on both sides of the rollers (turn the head over, rotate the rollers manually to clean the entire surface) (fig. 8).
  - c) The motorized flap (Don't mobilize the motorized flap) (fig 9 & 10).
- 3. Reattach the sealing flaps
- 4. Maintain the storage drawer using LPG wipes, then place the head in it.



Fig 10

# $\rightarrow$ MAINTENANCE

### **THE ALLIANCE 50 HEAD**


- 1. Remove the sealing flaps (2 up flaps and 1 down flap) as showed in the pictures below (fig 1 to 4).
- 2. Thoroughly rub for at least one minute with the wipes as describe here below: a) Flaps and their housing (fig 5 to 7).

b) The casing on both sides of the rollers (turn the head over, rotate the rollers manually to clean the entire surface) (fig. 8).

c) The motorized flap (Don't mobilize the motorized flap) (fig 9 & 10).

- 3. Reattach the sealing flaps
- 4. Maintain the storage drawer using LPG wipes, then place the head in it.



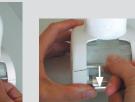

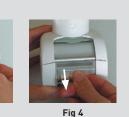


Fig 1

Fig 5





Fig 6





Fig 7

Fig 3

Fig 8





↗ ATTENTION Don't mobilize the motorized flap manually.

32

#### MAINTENANCE $\rightarrow$

### THE TR30 HEAD

Ι

HEADS

TREATMENT

- 1. Remove the sealing flaps as showed in the pictures below (fig. 1).
- 2. Thoroughly rub for at least one minute with the wipes as describe here below: a) The flaps and their housing (fig 2 & 3).
- b) The casing on both sides of the rollers (turn the head over, rotate the rollers manually to clean the entire surface) (fig. 4).
- 3. Reattach the sealing flaps
- 4. Maintain the storage drawer using LPG wipes, then place the head in it.

Fig 2









Fig 1

Fig 3

Fig 4

# $\rightarrow$ MAINTENANCE

### **ERGOLIFT HEAD AND ERGOLIFT CHAMBERS**

For reasons of hygiene, the maintenace of the care heads should be performed after each use, using antiseptic wipes impregnated with a bactericidal and fungicidal solution. Special attention must be given to the cleanliness of the parts that are in contact with the client.

- 1. Disconnect the chamber from the Ergolift care head. (fig. 1).
- 2. Remove the flap thanks to the dedicated tool. (fig. 2).
- 3. Thoroughly rub the Ergolift chamber, the flap and the tool for at least one minute with the wipes as describe here below. (fig. 3).
- 4. Put the flap back in the Ergolift chamber by following the same steps in the reverse order (fig. 4).



REMOVAL TOOLS

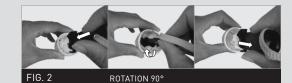






FIG. 4

↗ ATTENTION Don't mobilize the motorized flap manually.

# → ENDERMOWEAR<sup>™</sup>

LPG Endermowear<sup>™</sup> is available in several sizes for men and women and has been specially designed for body treatments. It is intended for personal use, guarantees hygiene, and its opaque areas cover the client's intimate parts during the treatment. Endermowear<sup>™</sup>'s unique material guarantees excellent adhesion to the skin which facilitates movement of the treatment head.

The products are delivered in a bag that the customer can personalize by putting their name on the label. It becomes the customer's property and can be used for several sessions. For esthetic reasons and hygiene, it should be washed after each use. Please refer to the washing instructions indicated on the bag label.

## → GENERAL WARRANTY CONDITIONS

You have recently acquired an appliance distributed by LPG Systems or an LPG Systems approved distributor. It is the purchaser/ user's responsibility to find out from the local authorities the conditions and professional qualifications required before using the appliance.

The purchase of this equipment implies the legal acceptance by the purchaser/ professional user of these general warranty conditions. If the appliance was sold to you by an approved LPG Systems distributor, the purchaser/ user should refer to the supplier's warranty conditions. These may in no way increase the undertakings made by LPG in these present warranty conditions.

The warranty can only be implemented and is only valid if the warranty slip has been duly filled out and returned to LPG Systems within two weeks of delivery, irrespective of the country. Warranty slips that are only partially completed will be rejected. The appliance is guaranteed against manufacturing flaws and defects in the raw materials.

The warranty extends for the shorter of the following two periods: one (1) year OR two thousand (2000) hours of use from the invoice date. During this period, LPG Systems undertakes to exchange or repair free of charge, as quickly as possible, any part that LPG Systems acknowledges as defective, however LPG Systems does not undertake to replace the entire appliance.

Traveling and living expenses for our technicians and transportation costs of the appliance or parts to and from the aftersales service workshop are not covered by this warranty. Replacements and repairs performed within this warranty, with or without immobilization of the equipment, shall not have the effect of extending the warranty period.

Replaced parts become the property of LPG Systems or the approved distributor. No compensation shall be paid for loss of use. Subject to other conditions hereafter, warranty shall apply if the purchaser/professional user has allowed LPG to proceed to necessary repair works.

# → GENERAL WARRANTY CONDITIONS

#### Warranty exclusion:

- Damage occurring during transportation. Transportation of this equipment and/or spare parts is at the recipient's own risk.
  Before taking delivery, it is the recipient's responsibility to verify the state of the goods and to make a claim against the transport company in the manner usual in the delivery country.
- Non-observance of the installation and operating instructions, failure to carry out maintenance and/or negligence in maintaining the appliance and/or its filter cartridges, connection to a faulty electricity supply or a non-earthed electricity supply or a power supply whose voltage is different to the one indicated on the appliance.
- If an appliance is sold before the end of the warranty period, the warranty is transferred to the purchaser for the remaining warranty period, on the condition that:
- I The original invoice is provided.
- **II** That the initial vendor is informed of the sale.

- Modification, mounting of accessories or dismantling of the equipment.
- Any operation and/or intervention not specified in the LPG Systems Operating Instructions and performed on the equipment by the purchaser/user and/or any party not approved by LPG Systems.
- Use of consumables, spare parts, inappropriate components or parts not supplied by LPG Systems.
- Blockage of the appliance through aspiration of a foreign body.
- Normal wear and tear of any of the equipment's parts resulting from normal usage.
- Damages or default resulting of accidental events (falls, impacts, ...)
  Damages or default resulting of natural disasters (lightening, water damages, ...)
  Fire, negligence or abuses.

### → GENERAL WARRANTY CONDITIONS

The warranty is limited to the replacement of the components of the device which comply with conditions described above. Under no circumstances shall LPG be liable for any loss or damage as a result or in connection with the device and/or its use, including any financial loss, loss of margin, loss of use, etc.

This clause shall apply under any and all legal basis. Whenever the above restriction may not be applicable or enforceable, LPG liability shall be limited to the price for the device and/or the service.

### → WARRANTY ACTIVATION

You can activate your warranty online by connecting to our warranty webpage: www.lpgsystems.com/warranty/

### → OR FILL IN THE FORM AND RETURN **TO LPG SYSTEMS**

~0
Name*:
Address*:
Country*:
Tel*:
Email:
Type of business:
Profession:
Type of unit: CELLU M6 ALLIANCE®
Serial number*:
Date:
The personal data collected in this form are sent to LPG Systems, which is responsible for processing it in order to manage customers relations, including after-sales service and warranties. Information with an asterisk must be provided in order for this form to be processed. According to modified French law 78-17 from January 6, 1978 relating to computers, digital files, and freedoms, also known as the "Informatique et Libertés" law, you have a right of access,

**~** 

rectify, and remove personal data about you by contacting LPG Systems in writing at the follo de la Plaine – 26000 VALENCE (FRANCE).

#### ATTENTION ■

To validate the warranty, detach this slip and return to the address below within 15 days following use of the unit:

LPG SYSTEMS® SA – Technoparc de la Plaine 30, rue du docteur Abel - CS 90035 - 26902 VALENCE Cedex 9 - France

### **HEADQUARTERS: LPG SYSTEMS**

TECHNOPARC DE LA PLAINE 30, RUE DU DR. ABEL - CS 90035 - 26902 VALENCE CEDEX 09 - FRANCE TEL.: +33 (0)4 75 78 69 00 - FAX: +33 (0)4 75 42 80 85

### INTERNATIONAL/MARKETING

2753, ROUTE DES DOLINES - BP 243 06905 SOPHIA-ANTIPOLIS CEDEX - FRANCE TEL.: +33 (0)4 92 38 39 00 - FAX: +33 (0)4 92 96 09 65

